



IMPORTANT: Please follow these instructions carefully when setting up your Mobilize Rescue Systems unit.



Mobile Rescue System | Setup Guide



You'll need some information before starting setup instructions:

1. **The Charging Cord** | you can find it in the plastic bag that came with the unit.
2. **Access to Wi-Fi** | The name of your Wi-Fi network and the Wi-Fi network password
3. Your Mobilize account name and password.
(If you don't have one yet we will create one later.)
4. Your Mobilize Comprehensive Unit Serial Number.
(It's the 8 digit number on the small white sticker on the back of the tablet.)

DISCLAIMERS. ALTHOUGH THE PRODUCTS MAY BE USED BY THE INDIVIDUAL TO PROVIDE MEDICAL ATTENTION, (A) THE MOBILIZE RRS, LLC IS NOT A HEALTH CARE PROVIDER AND IS NOT RENDERING MEDICAL ADVICE OR DIAGNOSIS TO ANYONE; AND (B) THE PRODUCTS ARE NOT INTENDED TO REPLACE THE NEED FOR SEEKING MEDICAL ATTENTION FROM A MEDICAL PROFESSIONAL.



1

Let's make sure the battery is fully charged.

1. Remove the tablet from the touch-through pocket in the case lid.
2. Assemble the cord and plug in the tablet.
3. Keep going while the unit is charging.

2

Connect to Wi-Fi.

4. Locate **Wi-Fi Settings** on the right side of the screen and touch **On/Off**.
5. On the next screen find **Off** near the top and toggle it to **On**. 
6. Choose your WiFi network, enter your WiFi network password and touch **Connect**.
7. Touch the **Back button**  twice on the bottom of the screen.




3

You are at the sign in page. Let's get you registered.

8. Sign in if you have an account, or choose Create Account and follow the prompts.
9. Enter your serial number. (It's the 8 digit number on the back of the tablet.)

4

Almost done, we just need to make sure Wi-Fi is turned off.

10. On the top right, touch the little red gear icon. 
11. Locate **Wi-Fi Settings** and touch **On/Off** on the right side of the screen.
12. On the next screen find **On** near the top and toggle it to **Off**. 
13. Touch the **Back button**  twice on the bottom of the screen.
14. You will be at the **TAKE A DEEP BREATH** screen.
15. **The battery is in upper right corner. At 100% unplug the tablet and replace it in the touch-through pocket in the lid so the red dot is in the lower right corner.**

5

Good job!

Your Mobile Rescue System is ready for use. Close your Mobile Rescue System. Your unit will go to sleep after several minutes of inactivity.



YOU MUST CHARGE YOUR UNIT ONCE A MONTH.

You will receive monthly emails reminding you to charge your unit, perform a diagnostic check, check expiration dates, and more.

We'll let you know by email if you need to enable WiFi to update your software.

More information

For more information go to webportal.mobilizerescue.com and sign in with the email and password you created during the registration process.

Under the Mobile tab you will find:

- Introduction videos
- Planogram and unit contents
- Manufacturer recommendations for monthly diagnostic checks
- Recommendations for inspection cards and proof seals

Replacing used or expired supplies

Go to webportal.mobilizerescue.com, sign in and choose "Store" from the menu.

For setup assistance contact **Mobilize Rescue Systems Customer Support** at **585.635.7260**
8am - 6pm Eastern Time or **support@mobilizerescue.com** for a reply within 24 hours.



@mobilizeRescue



@mobilizeRescue



Mobilize Rescue Systems



Mobilize Rescue Systems



585.635.7260



support@mobilizerescue.com

© 2017 Mobilize Rescue Systems. All rights reserved.

www.mobilizerescue.com